



## PREPARATION



Wash your hands and sit in a chair with low backrest.

Put the Unit dose vial, top first, into the UD **Eyot**<sup>®</sup>.



Click Unit dose vials end to the UD **Eyot**<sup>®</sup>.



Twist off Unit dose vials top.



Hold the UD **Eyot**<sup>®</sup> so that the hole is above the Unit dose vial.

## TO PLACE THE UNIT DOSE VIAL IN FRONT OF YOUR EYE



Place the lower part of the UD **Eyot**<sup>®</sup> under your eye and pull the eyelid gently downwards.



Place the upper part of the UD **Eyot**<sup>®</sup> near your eyebrow against your forehead. You'll see the Unit dose vials dropper as a white dot in front of your eye.

## TO PLACE YOUR HEAD IN THE CORRECT POSITION



Look for the ceiling straight above you through the hole.

Squeeze the Unit dose vial gently until 1 to 2 drops as required drips in to the affected eye.

Read the package insert provided with your medication.



## CLEANING ADVICE

Clean the UD **Eyot**<sup>®</sup> daily by rinsing it in lukewarm water when replacing the Unit dose vial.

Dry the UD **Eyot**<sup>®</sup> with a clean cloth.

Do not use hot or boiling water. Do not clean the UD **Eyot**<sup>®</sup> in the dishwasher.

## OTHER INFORMATION

The UD **Eyot**<sup>®</sup> should only be used with Celluvisc Unit dose vials and with Liquifilm Tears unit dose vials. Do not use a damaged UD **Eyot**<sup>®</sup>.

Take your medication as instructed on the package leaflet or as prescribed by your doctor if different.

The UD **Eyot**<sup>®</sup> is made of a thermoplastic resin material and is recyclable.

**Manufacturer:**  
Multidee Inventors  
PO Box 135  
4130 EC Vianen  
The Netherlands

**Distributeur:**  
Allergan Limited  
Coronation Road  
High Wycombe  
Buckinghamshire  
HP12 3SH

1090-0001-00 100206

Celluvisc<sup>®</sup> and Liquifilm Tears Unit Dose<sup>®</sup> are registered trade name. **Eyot**<sup>®</sup> is a registered trade name.

Patents applied and pending.



## INSTRUCTIONS FOR USING THE UD **Eyot**<sup>®</sup>



An eye drop aid for administering Unit Dose eye drops of Allergan.

Please read this leaflet carefully.