

1. Preparation



Wash your hands and sit in a chair with a low backrest.

Push the closed bottle, base first, downwards into the device until it is firmly in position.

Check before each use that the bottle is firmly in position in the device.



Unscrew the lid.

Take the device in your hand.

You should hold the device so that the mirror is on the lower side (see photo).

2. To place the bottle in front of your eye



Place the lower part of the device just below your lower eyelid and pull the eyelid gently downwards until there is a "pocket" between the eyelid and your eye (see photo).



Place the upper part of the device near your eyebrow, against your forehead.

Look for your eye in the **Eyot**'s mirror. You may find this easier if you shut/cover your other eye.

The device and the bottle are now in the correct position.

3. To correctly position your head



Now look up through the hole in the upper part of the device.

Tilt your head backwards until you can see the ceiling directly above you, through the hole.

Your head is now in the correct position.

Squeeze the device carefully with your fingers. The eye drop should fall directly into your eye.

You should repeat the process for your other eye, if directed by your doctor.

Close the bottle again with the lid. You can keep the bottle in the device.

See overleaf for cleaning instructions.

Accessoiry (optional)



Push the wide side of the **Eyot**® Bottle Opener on the lid of the bottle, and twist the lid off.

Pay attention that the bottle remains in the aid properly.

You can leave the **Eyot**® Bottle opener on the bottle's lid.

Instructions for using the Azopt®Eyot®



Your ophthalmologist has provided you with this device to help you use your Azopt® eyedrops.

Manufacturer: 1030-0003-00 20-05-2005
Multidee Vianen
PO Box 135, 4130 EC Vianen, the Netherlands.

Distributor:
Put here your address.

CE Azopt® is a registered trade name.
E Eyot® is a registered trade name.
Patents pending.

Azopt®Eyot®

Carefully read these instructions before using the Azopt®Eyot® device.

It is important that you use your medicine regularly as recommended by your doctor. Further information about the use of Azopt® eyedrops can be found on the patient information leaflet enclosed with each Azopt® bottle.

Cleaning advice

Leave the bottle in the device and wipe the device with a clean dry cloth.

If necessary remove the bottle and rinse the device in lukewarm water. Dry it with a clean cloth. Do not use hot or boiling water. Do not clean the device in a dishwasher.

Other information

Check before each use that the bottle is firmly positioned in the device.

The Azopt®Eyot® should only be used in combination with Azopt® eyedrops. This device provides a simple, alternative, instillation technique.

You can keep the Azopt® bottle in the Azopt®Eyot®.

Azopt®Eyot® is made of a recyclable thermoplastic synthetic material. The mirror is made of a metallic polyester.

Warning



Maximum height

The moulded line marks the maximum height of the bottle.

Make sure that the bottle remains under this line.

This device is only for use with Azopt® 5 ml bottle of Alcon Laboratories.

To avoid possible injury, please ensure that the bottle is pushed firmly into the device.

In case you use the (optional) Eyot® Bottle Opener, please ensure that the bottle remain at its place into the device. Do not pull bottles' lid off unless you have screwed off the bottles' lid completely.